# Précise 1100S

<b>(DE) Gebrauchsanweisung .....</b>	<b>2</b>
<b>(GB) Instructions for Use .....</b>	<b>16</b>

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Edition: Rev. G 2024-08  
Date of issue: 2024-08-05

# Table of Contents

<b>Content</b>	<b>Page</b>
<b>1 Introduction .....</b>	<b>18</b>
1.1 Information about this document .....	18
1.1.1 Importance of Attention and Caution .....	18
1.2 Intended use .....	18
1.2 Functional description .....	19
1.3 Important use and safety instructions .....	19
<b>2 Operation .....</b>	<b>21</b>
2.1 Switching on the device .....	21
2.3 Oxygen concentration measurement.....	22
2.4 Flow measurement .....	23
2.5 Pressure measurement.....	23
2.6 Switching off the device .....	23
2.7 Charging the battery .....	24
<b>3 Cleaning .....</b>	<b>24</b>
<b>4 Maintenance .....</b>	<b>25</b>
<b>5 Disposal .....</b>	<b>25</b>
<b>6 Technical Data.....</b>	<b>26</b>
6.1 General .....	26
6.2 Definition of the pictograms located on the device .....	26
6.2.1 At the connections .....	26
6.2.2 Label .....	27
6.3 Measurement parameters.....	28
<b>7 Scope of delivery .....</b>	<b>29</b>
<b>8 Warranty.....</b>	<b>29</b>

# 1 Introduction

## 1.1 Information about this document

Precondition for the use of the Précise 1100 S is the exact knowledge and observance of these instruction for use. Read all safety instructions and warnings carefully before using the device. This document must be kept as long as the Précise 1100 S is in use.

### 1.1.1 Importance of Attention and Caution

The expressions CAUTION and ATTENTION are used in this document in connection with the symbols at various passages.

**ATTENTION:** Important safety instructions that can lead to an immediate defect or even a fire if not observed.

**CAUTION:** Instructions which, if not observed, will result in damage to the measuring

## 1.2 Intended use

The Précise 1100 S is a battery-operated measuring device. It is used for percentage determination of oxygen content, flow measurement and pressure measurement on an oxygen concentrator, which enriches the inhalation gas with oxygen by pressure swing absorption and zeolite.

The Précise 1100 S must not be used to monitor life support systems.

Precondition for the intended use of the Précise 1100 S measuring instrument is the exact knowledge and observance of these operating instructions.

The use of accessories that have NOT been approved may result in impaired performance.

Only original parts are to be used. Worn or damaged parts such as the charger, charging cable or housing must not be used.

## 1.2 Functional description

An ultrasonic sensor is used for oxygen concentration and flow measurement. The oxygen concentration measurement and the flow measurement are based on transit time measurements of a sound wave. The pressure measurement is carried out by means of a piezoelectric pressure sensor.

## 1.3 Important use and safety instructions

Do not replace the energy cells. Otherwise, there is a possibility of fire, which is much more dangerous in connection with an oxygen-enriched air, and often occurs earlier.

Do not open the device.

If a malfunction is suspected, return the device to medicap homecare GmbH and do not perform any measurements on an oxygen concentrator.

Charge only with the supplied charger.

Load on a non-flammable surface.

Charge at ambient temperatures between 10 and 30 °C Celcius

Do not charge unattended and do not charge longer than necessary.

Do not store fully charged or fully discharged.

An annual inspection of the charger in accordance with the DGUV is the responsibility of the operator in the commercial environment.

The Précise 1100S and all accessories that come into contact with oxygen must be kept free of oil and grease.

Only use original parts from medicap homecare GmbH.

Smoking, naked light and fire are prohibited during usage. Protect from moisture and humidity.

The Précise 1100 S provides optimum accuracy with oxygen concentrators that use pressure swing absorption and zeolite molecular sieves.

Protect from sunlight and keep away from heat sources.

No humidifier may be connected between the oxygen concentrator and the Précise 1100 S.

If the device has been stored at temperatures below 10°C, the device must adjust to room temperature, otherwise malfunctions may occur. Only measure at room temperature.

---

---

Maintenance on the Précise 1100 S must be performed annually by medicap homecare GmbH or authorized personnel.

---

---

If the displayed measured values are completely doubtful, send the device immediately to an authorized service technician or medicap homecare GmbH.

---

---

If an error message such as Error E1, E2, E3 or the setup menu (## SETUP ##) appears after switching on, send the device immediately to an authorized service technician or medicap homecare GmbH.

---

## 2 Operation

### 2.1 Switching on the device

Before switching on for the first time, the Précise 1100 S should be charged using the charger. To do this, connect the charger and the Précise 1100 S.

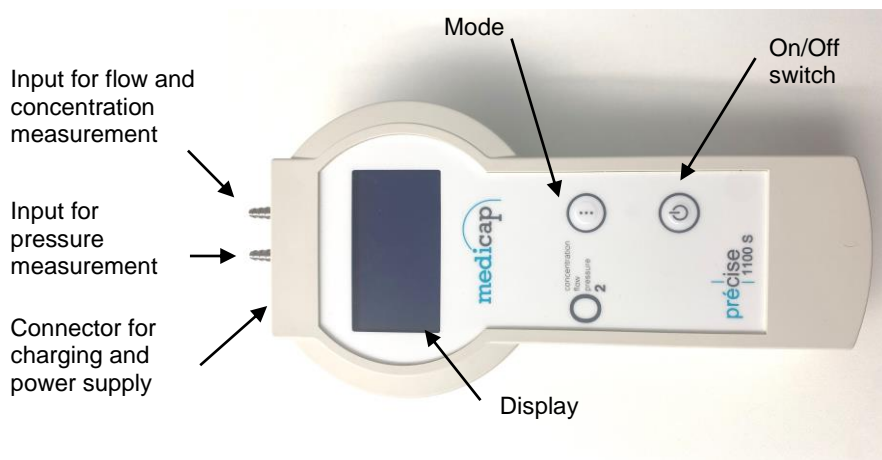
Fig. 1: Front view of the Précise 1100 S

To switch on, press the on/off switch.

After switching on, the device performs a self-test. It is then ready to measure.

If an error message such as Error E1, E2, E3 or the setup menu (## SETUP ##) appears after switching on, send the device immediately to an authorized service technician or medicap homecare GmbH.

The oxygen concentration measurement is displayed first. The display illumination switches off after approx. 20 seconds.

Briefly pressing one of the buttons reactivates the illumination for 20 seconds. A long press on the MODE button also activates the display lighting for 20 seconds and changes the measuring mode at the same time.

## 2.2 Switching the display modes

On the Précise 1100 S, the measured value can be displayed both numerically and graphically.

The display mode can be selected using the Mode button.

Press the Mode button	Display
0; 7	numerical display of the oxygen concentration ( vol.% )
1	numerical display of the flows ( l/min )
2	numerical display of the pressure value ( bar ) or mbar
3	numerical display of the pressure value ( psi )
4	numerical and graphical display of oxygen concentration ( vol.% )
5	numerical and graphical display of flow ( l/min )
6	numerical and graphical display of the pressure value ( bar/ mbar )

## 2.3 Oxygen concentration measurement

No humidifier may be connected between the oxygen concentrator and the Précise 1100 S.

After switching on, the display shows the current oxygen concentration numerically. To display the oxygen concentration numerically and graphically, the Mode button must be pressed 4 times.

Now set the concentrator to the desired flow. Connect the output of the oxygen concentrator to Input for flow and concentration measurement on the Précise 1100 S using the tubing. The oxygen concentration can be read in the display. Please note that with most oxygen concentrators an accurate concentration measurement can only be taken 10 minutes after the device has been switched on.

## 2.4 Flow measurement

No humidifier may be connected between the oxygen concentrator and the Précise 1100 S.

Connect the output of the oxygen concentrator to the Input for flow and concentration measurement on the Précise 1100 S using a tubing. After selecting the display, the flow can be read on the display. To display the flow numerically and graphically, press the Mode button 5 times.

## 2.5 Pressure measurement

No humidifier may be connected between the oxygen concentrator and the Précise 1100 S.

To measure pressure, the pressure line must be connected to the pressure port (connection) on the Précise 1100 S.

The pressure can be displayed in bar or psi (see 2.2. Switching display modes). For numerical and graphical display of the pressure, the Mode button must be pressed 6 times

For prolonged pressure measurement, the tubing should be secured to the measuring device and the measuring connection.

## 2.6 Switching off the device

Press the on/off button again to switch the machine off. The Précise 1100 S switches itself off 5 minutes after the last button has been pressed. An acoustic warning signal sounds after 4 minutes.

## 2.7 Charging the battery

- 
-  Only charge with the charger supplied.

---

  -  Load on a non-flammable surface.

---

  -  Charge at ambient temperatures between 10 and °C degrees Celcius.

---

  -  Do not charge unattended and do not charge longer than necessary.

---

  -  Do not store fully charged or fully discharged.

---

  -  Protect from sunlight and keep away from heat sources.

---

  -  An annual inspection of the charger in accordance with the DGUV is the responsibility of the operator in the commercial environment.

---

To charge, the charger is connected to the Précise 1100 S.

A charging cycle can take up to 9 hours.

The expected battery runtime in measuring mode is approx. 7 hours. However, by using the display illumination, this runtime can be significantly undercut.

A self-switch-off, if no button is pressed, takes place after 5 minutes. An acoustic warning signal sound after 4 minutes.

## 3 Cleaning

The device should only be cleaned with a dry cloth.  
No liquid may get into the device.

## 4 Maintenance

---

**Caution**

Maintenance on the Précise 1100 S must be performed annually by medicap homecare GmbH or authorized personnel.

---

**Caution**

Works on the opened device may only be performed by an authorized service technician.

---

**Caution**

All maintenance intervals are guide values. In individual cases, these may be shorter due to changes in environmental conditions with regard to temperature, humidity, dust, etc.

---

**Caution**

During maintenance, only monitored test equipment and measuring devices suitable for the intended purpose may be used.

---

**Caution**

If the displayed measured values are completely doubtful, send the device immediately to an authorized service technician or medicap homecare GmbH.

---

If maintenance is carried out to the prescribed extent and at the specified intervals, the device will operate safely during the intervening period.

## 5 Disposal

The device and its packaging can be returned to medicap homecare GmbH for disposal free of charge. We advocate environmentally friendly disposal procedures.

The LiFePo batteries contained in the device must be returned to medicap homecare GmbH for environmentally sound disposal.

## 6 Technical Data

### 6.1 General

Dimension (incl. connections): 250 mm x 115 mm x 45 mm

Weight (incl. battery without protection): 436 g

Power supply: internal 3x 3,2V 600mAh LiFePo4 rechargeable batteries, external 15 V DC battery charger

Display: graphic LC display, blue backlight

### 6.2 Definition of the pictograms located on the device

#### 6.2.1 At the connections

	Connector for charging and power supply
	Input for pressure measurement
	Input for flow and concentration measurement

## 6.2.2 Label

	Caution
	Item number
	Serial number
	Unique device identification
	Date of manufacture
	Manufacturer
	Protect against moisture
	Protect from direct sunlight
	CE marking The CE marking confirms conformity with the applicable directives
	Fragile
	Observe instructions for use
	The device must not be disposed of with household waste, see Chap. 4
	IEC for barrel connector

### 6.3 Measurement parameters

Measured variable	Measuring range	Measuring accuracy	Additional note
Pressure measuring range:	0 – 2,0 bar 0 – 150 mbar 0 – 29 psi	+/- 1% +/- 1% +/- 2%	
Flow:	1-- 10 l/min  > 10 l/min	+/- 0,2 l/min or +/- 5% which is bigger  Not specified	*1
Oxygen concentration:	21 – 95,6 vol.%	+/- 2,5 %	

\*1) at 25°C +/- 2 °C medium temperature

## 7 Scope of delivery

Précise 1100 S complete, consisting of:

1x Measuring device incl. battery	Art.-no. 221.010
1x Measuring tubing	Art.-no. 221.003
1x Instructions for Use	Art.-no. 221.004
1x Charger	Art.-no. 221.424

## 8 Warranty

From the date of delivery, we grant a two-year guarantee for defects that are due to material or manufacturing faults. Defects covered by the warranty claim will be repaired within the scope of our warranty conditions. Furthermore, medicap homecare GmbH does not provide any warranty if the operator endangers the functions of the device by not observing these instructions for use, by not using the device as intended or by third-party intervention. In these cases, liability is transferred to the operator.

### Important

The warranty can only be claimed in connection with the proof of purchase.

medicap homecare GmbH  
Hoherodskopfstr. 22  
35327 Ulrichstein  
Germany

Tel.: +49 / 06645 / 970-0  
Email: homecare@medicap.de